

Maymom IBCLC Starter Kit

Thank you for choosing Maymom IBCLC Starter Kit (ISK). We are willing to help anyone who intends to become a lactation consultant. This kit contains connectors and shields of many sizes for finding the best fit for the mothers. Also, there are various adapters which can combine breast pump parts together even from different brand makers. Following is detailed information:

Kit Contents

Model No. IBCLC-StarterKit_A

PART 1. Base Connectors and Valves & Membranes

PART 2. Maymom MyFit Shields and Flange Inserts

PART 3. Adapters

PART 4. Maymom Pump Kit Set for speCtra Pump

PART 5. Other tools

- Nipple Ruler Card – for measuring moms' nipple diameter
- Feeding Cup – for baby needing short-term feeding

Cleaning Guide

It is important to ensure the parts are clean and disinfected, especially when different users and human milk contact the parts. Made of various materials, each accessory has its own cleaning method. Please refer to the respective instructions and follow them accordingly.

Before use and once daily

- **DO NOT wash the Tubing**
- Disassemble all other parts, clean with soapy water and rinse in warm water.
- Boil the parts in boiling water for 3-4 minutes.

After every use

- **DO NOT wash the Tubing.** Air dry. Replace the Tubing if it gets dirty.
- Disassemble, clean and rinse the parts in warm water.
- Replace the duckbill if it has a gap more than 0.5 mm (thickness of 5 pieces of copy paper) at the lip area.



Make sure you have plenty of water in the pot.



The rim of the pot without water will become over heated and it will melt the plastic.

Base Connectors

PART 1

- Separable connector for breastshield.
- Maymom MyFit Shields from PART 2 are needed to make the connection.



Model No. M012-tBx2

- **Narrow-neck** base connector.
- Top air path type.
- Compatible with Medela Shields, Maymom shields.
- Suitable for Medela pumps made prior Jun 2020 - Pump in style, Swing, Lactina, Symphony.



Model No. A012-Bx2

- **Narrow-neck** base connector.
- Interchangeable and compatible with Ameda breastshields.
- Suitable for Ameda pumps: Purely Yours, Purely Yours Ultra, Elite, Mya Joy, Finesse, and Platinum.
- Compatible Pumping Bra: Simple Wishes, medela, BRAVADO, Pump & Nurse, Pacasso, Pump Strap.



Model No. S012W-Bx2

- **Widemouth** base connectors.
- Compatible with PersonalFit Shields, Pumpin Pal Breast Pump Flange, Lansinoh ComfortFit Breast Pump Flanges, and Maymom MyFit shields 10-36 mm.
- Suitable for speCtra pump S1, S2, 9 plus.
- Handsfree pump bra friendly.
- Works with speCtra wide mouth bottles, AVENT bottles, and Maymom wide mouth bottles.

Valves & Membranes



Model No. M005-4V6M



- Compatible with Medela Breast pumps: Swing, Lactina, Pump in Style.

Notice - DO NOT let the breastmilk dry out between the valve and breastshield. Dried milk can act like glue!

Maymom MyFit Shields

PART 2

- Use with Base Connectors from PART 1.
- Compatible with Medela pumps: New Medela Pump in Style with MaxFlow, Pump in style, Lactina, Symphony, Swing, Mini Electric FreeStyle, Swing Maxi, Sonata, Freestyle Flex and Harmony.
- Easy to detach and attach for thoroughly cleaning.
- Work with all hands-free pumping bras.

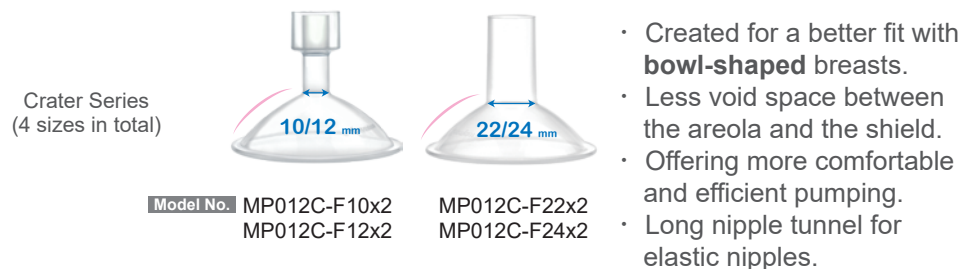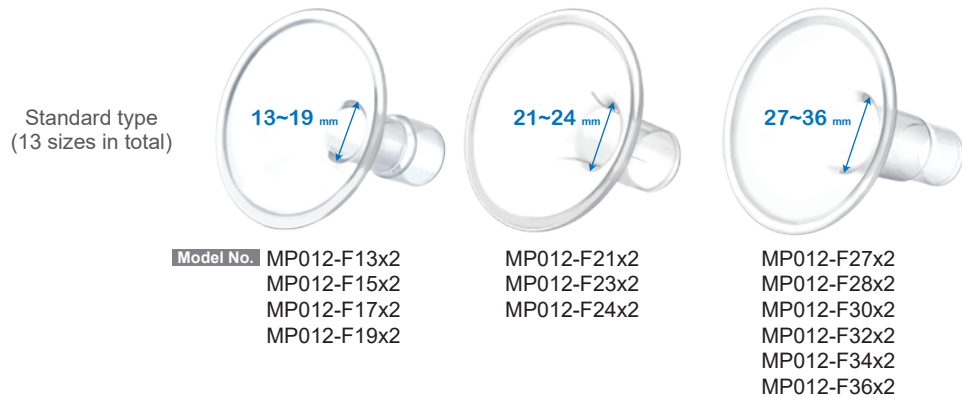


Flange Inserts





Adapters

PART 3

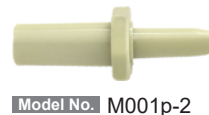


- Connects between a Medela PIS MaxFlow/Freestyle Flex connector and speCtra pump S1/S2. (See graph 1-1)
- Connects between a maymom narrow neck flange/connector and a bare end tubing. (See graph 1-2)

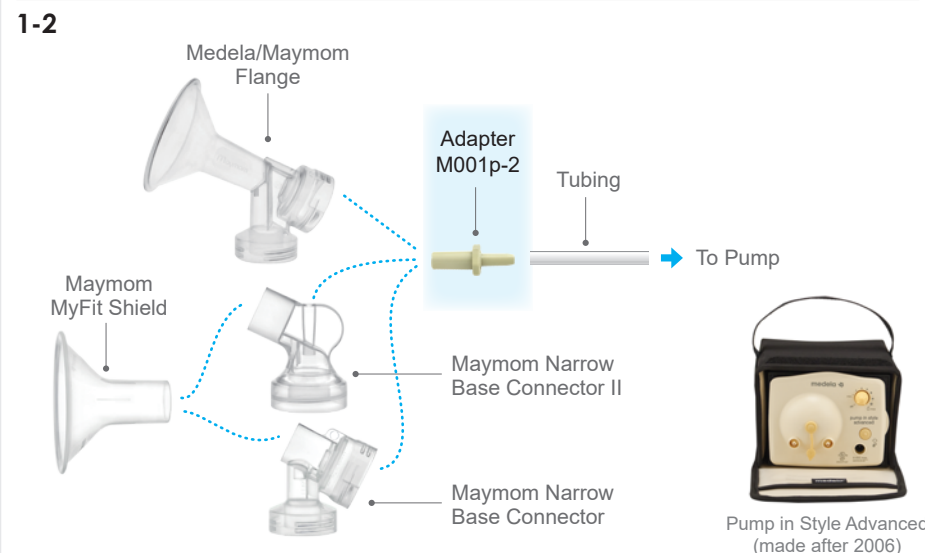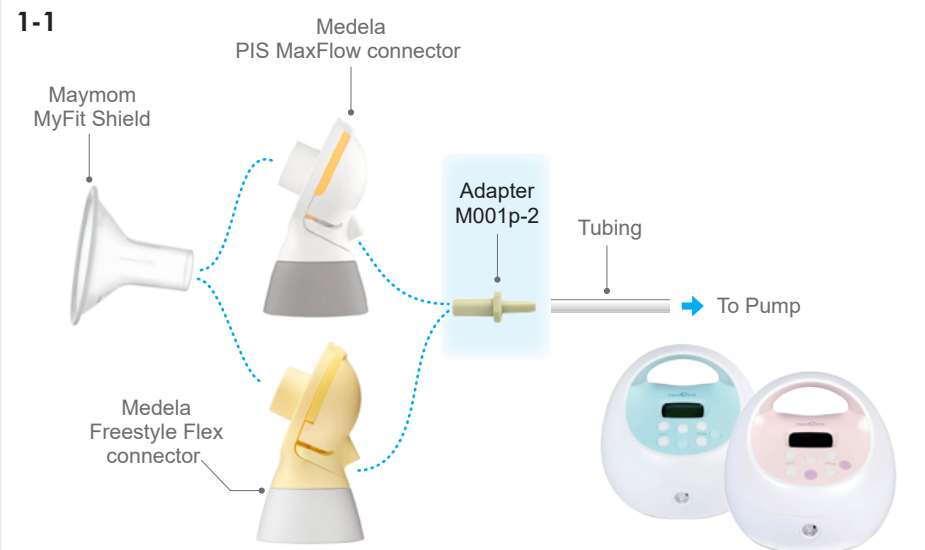

Adapters

PART 3

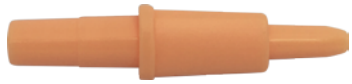




Model No. M030p-2

- Connects between a Medela Sonata connector and speCtra pump S1/S2. (See graph 2-1)
- Connects between a Maymom widemouth flange/connector and a bare end tubing. (See graph 2-2)



Model No. M002p-2


- Connects between a Medela Lactina, Symphony, Swing, as well as Pump in Style Advanced (made between 2004 ~ July 2006) pump and a bare end tubing. (See graph 3-1)
- Connects between a Medela Swing Maxi pump and a bare end tubing. (See graph 3-2)

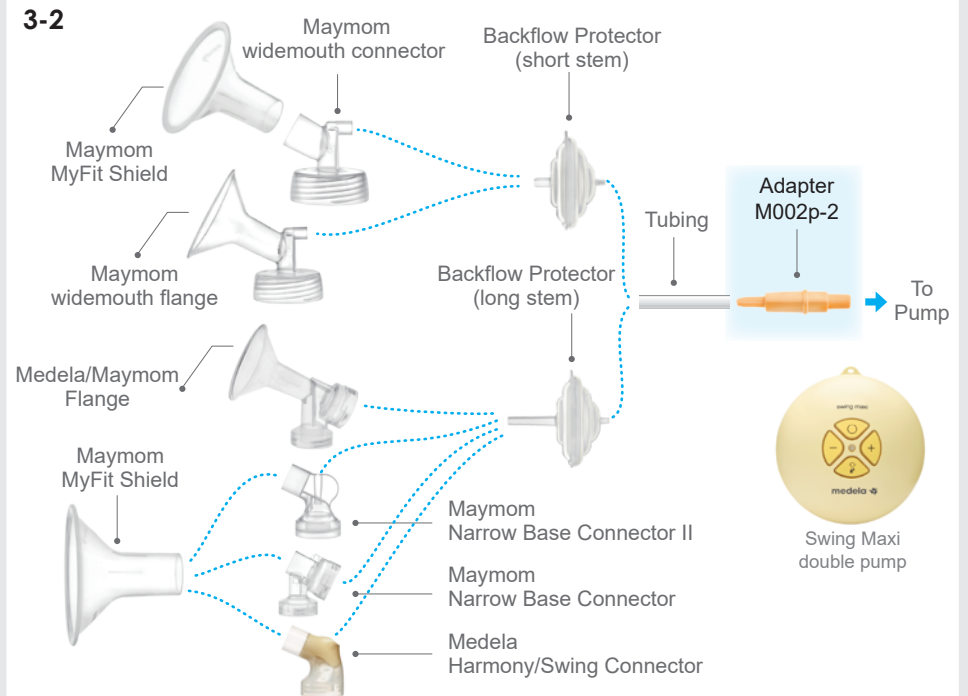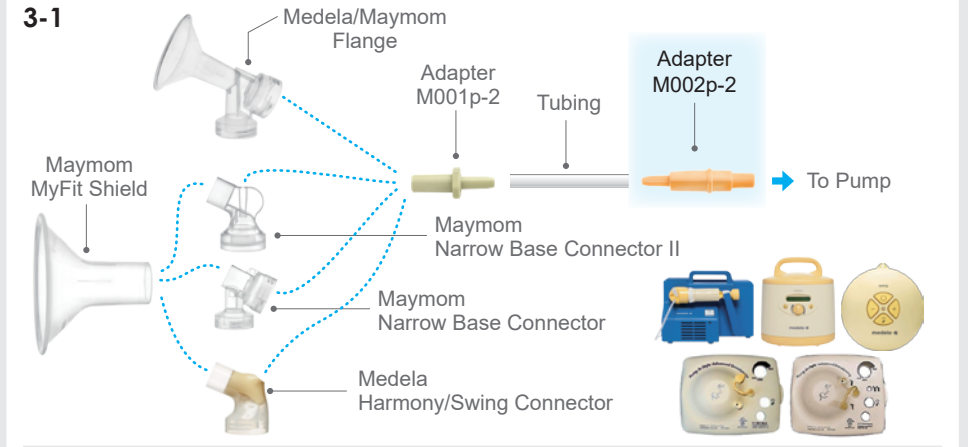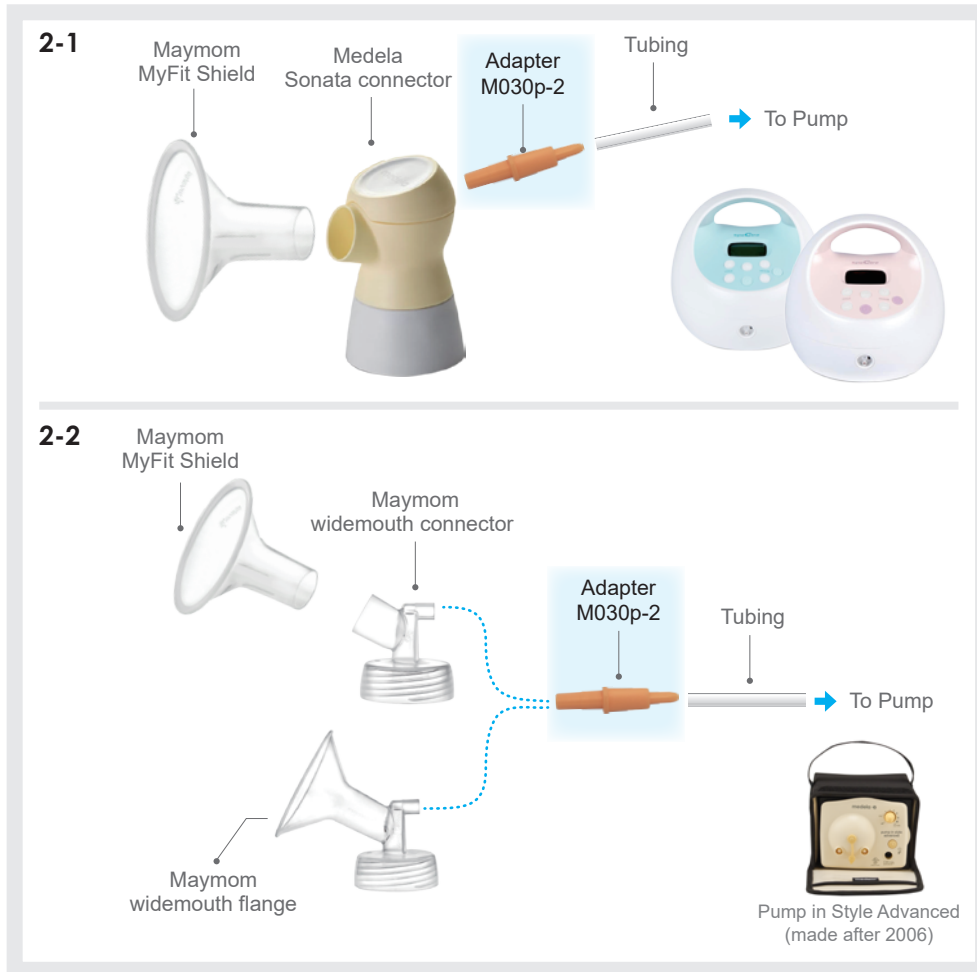




Model No. M011p-2

- Connects between Medela Swing Maxi connector and speCtra pump S1/S2. (See graph 4-1)
- Connects between a Medela Freestyle (old version) connector/pump and a bare end tubing. (See graph 4-2)

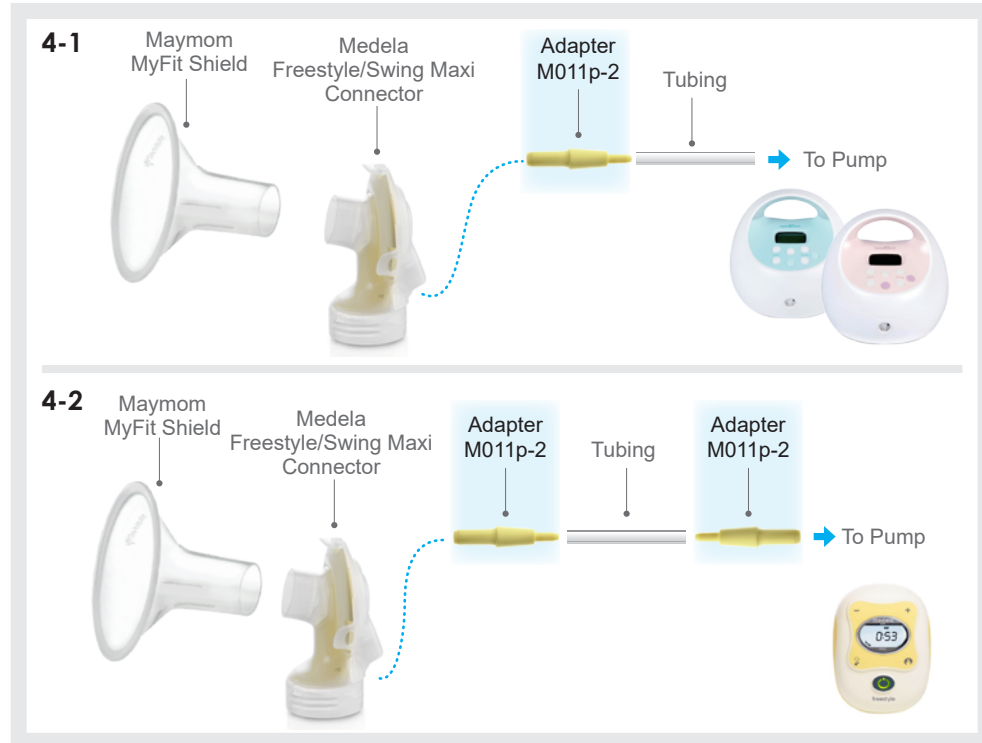


Model No. S012p-2

- For using Medela flanges with speCtra pump S1, S2, M1, 9 Plus. (See graph 5-1)

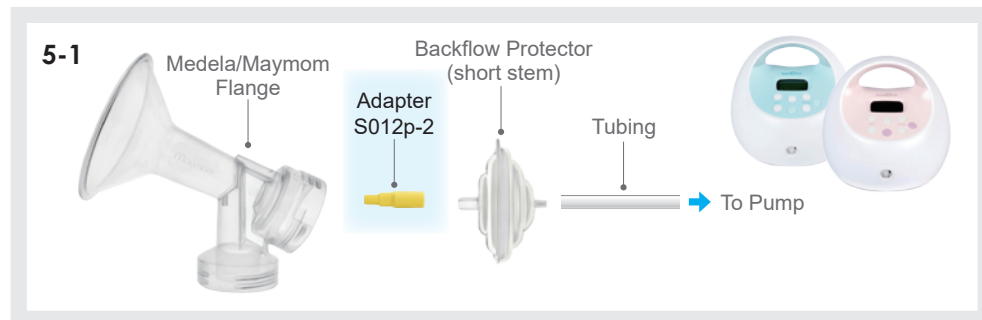

Model No. FP002-4Sc

- Can be used in many applications where 3.2 - 4.0 mm size change to 6.2 - 8.5 mm is needed. (See graph 6-1 & 6-2)
- Size changers for Freemie cups to be used with other pumps. (See graph 6-2)

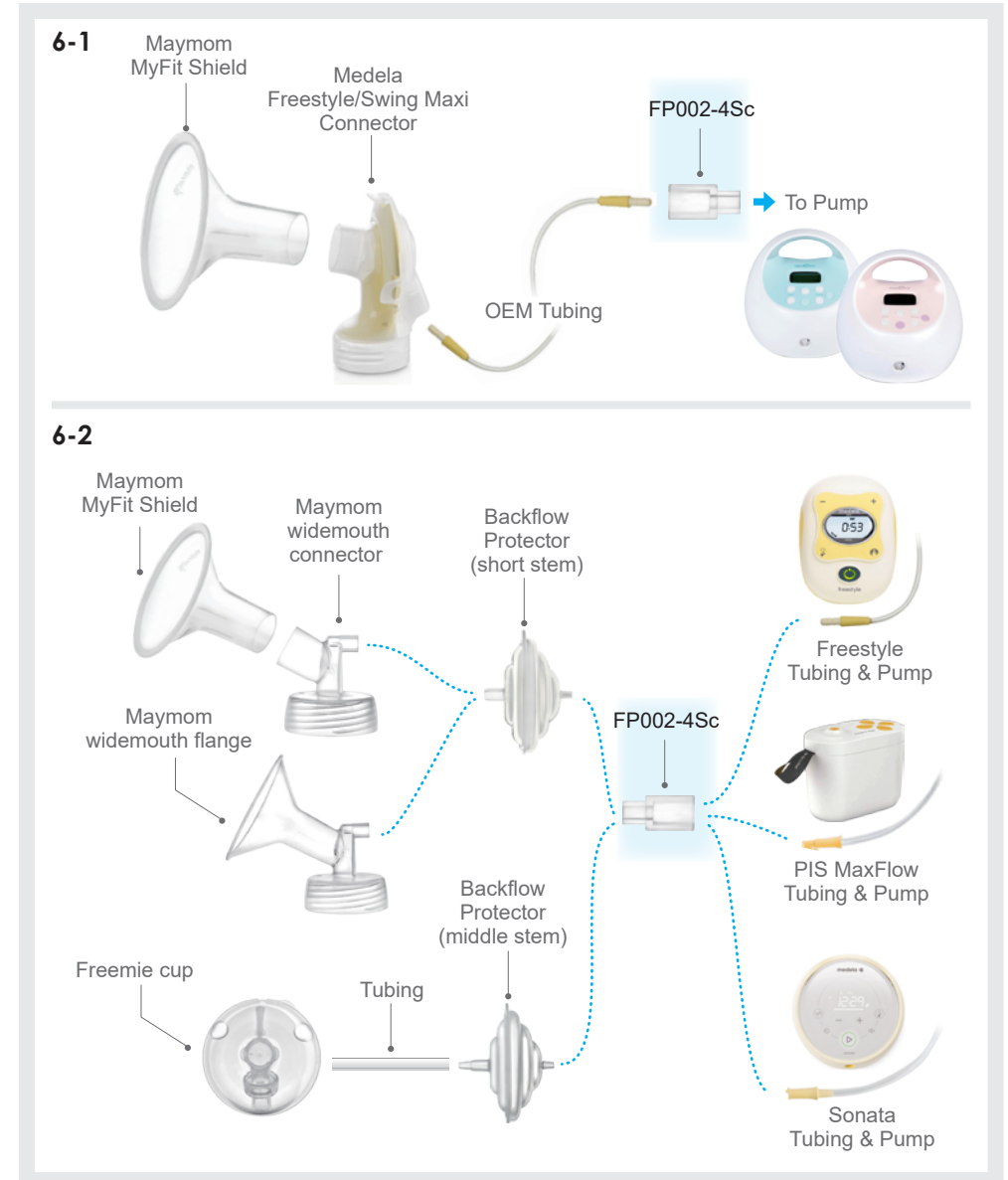

Maymom Pump Kit Set for speCtra Pump PART 4



- Work with speCtra pump S1, S2, 9 Plus
- This kit includes:
 - 2 Flanges (24 mm)
 - 2 Duckbill Valves
 - 2 Backflow Protectors (short stem)
 - 2 Tubes
 - 2 Bottles with One-Piece Travel Caps.
- Multiple flange sizes available: 15/17/19/21/24/25/27/28/30/32 mm

Model No. S001W-F24.5in1B



Nipple Ruler Card PART 5


Nipple Diameter Measuring Tool

- 1 Cut through the red line and bend along the green dots.
- 2 See the photo to the left for a finished tool
- 3 Measure the nipple diameter AFTER a pumping session.

- 4 Start to try a breastshield that is 1-2 mm larger in internal ID than your nipple diameter.

Cut through the red line
Choosing the right size of breastshields is like choosing shoes to fit your feet

- A tool for moms to measure their nipple diameter.
- Remember to measure the nipple diameter **AFTER** a breast pumping session. Choose a breastshield with an internal diameter that is 1-2 mm larger than your nipple diameter.
- If you measure **BEFORE** a pumping session, you can only estimate to get one that is close. Try one that is 3-5 mm larger than your nipple diameter.

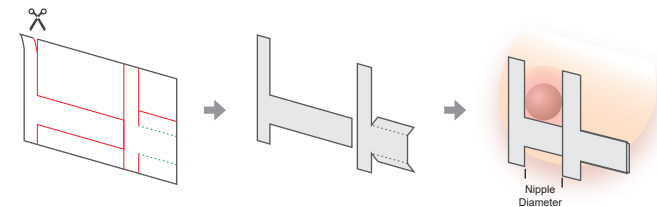

Feeding Cup



Model No. X225-4Cup

- Easy to track the small amount of feeding.
- For babies requiring medication or supplement
- For premature babies
- Babies with sucking disorder such as cleft lip/palate children
- Alternative feeding method to bottle or spoon feeding

Contents List

Model No. IBCLC-StarterKit_A

| Model No. | Quantity | Model No. | Quantity | Model No. | Quantity |
|--------------|----------|--------------|----------|-----------------|----------|
| A012-Bx2 | 1 pack | MP012C-F24x2 | 1 pack | MP012-F34x2 | 1 pack |
| F012-i21x2 | 1 pack | MP012-F13x2 | 1 pack | MP012-F36x2 | 1 pack |
| FP002-4Sc | 1 pack | MP012-F15x2 | 1 pack | Nipple Ruler | 20 pc |
| M001p-2 | 1 pack | MP012-F17x2 | 1 pack | S001W-F24.5in1B | 1 pack |
| M002p-2 | 1 pack | MP012-F19x2 | 1 pack | S012p-2 | 1 pack |
| M005-4V6M | 1 pack | MP012-F21x2 | 1 pack | S012W-Bx2 | 1 pack |
| M011p-2 | 1 pack | MP012-F23x2 | 1 pack | X012-i15x2 | 1 pack |
| M012-tBx2 | 1 pack | MP012-F24x2 | 1 pack | X012-i17x2 | 1 pack |
| M030p-2 | 1 pack | MP012-F27x2 | 1 pack | X012-i19x2 | 1 pack |
| MP012C-F10x2 | 1 pack | MP012-F28x2 | 1 pack | X225-4Cup | 1 pack |
| MP012C-F12x2 | 1 pack | MP012-F30x2 | 1 pack | | |
| MP012C-F22x2 | 1 pack | MP012-F32x2 | 1 pack | | |

Maymom Customer Service

✉ CS@maymom.com

🏠 <http://maymom.com/>

☎ +1 919 4535168

📍 Maymom, LLC

3 Laurel Leaf CT Durham, NC 27703



Maymom always cares about mothers' needs on the path of breastfeeding. For years, we hear mothers' voice for customized, comfortable and easy-to-use tools. Maymom products feature safety, efficiency and credibility that you can count on.



Find more!

*Ameda is a registered trademark of Ameda INC. Corp.
Maymom is not affiliated with Ameda nor does Ameda endorse Maymom's products and compatibility claims.

*Avent is a registered trademark of Koninklijke Philips N.V..
Maymom is not affiliated with Philips nor does Philips endorse Maymom's products and compatibility claims.

*Freemie is a registered trademark of Dao Health.
Maymom is not affiliated with Dao Health nor does Dao Health endorse Maymom's products and compatibility claims.

*Medela is a registered trademark of Medela Holding AG, Switzerland.
Maymom is not affiliated with Medela nor does Medela endorse Maymom's products and compatibility claims.

*SPECTRA and the SPECTRA Logo are registered trademarks of Mother's Milk, Inc. DBA Spectra Baby USA and Min Byung Wook DBA Uzinmedicare.
Maymom is not affiliated with them nor do they endorse Maymom's products and compatibility claims.